

Getting Started




Infrared Thermometer

STEP 1

Device Setup

Insert 2 x AAA batteries into bottom of the unit.

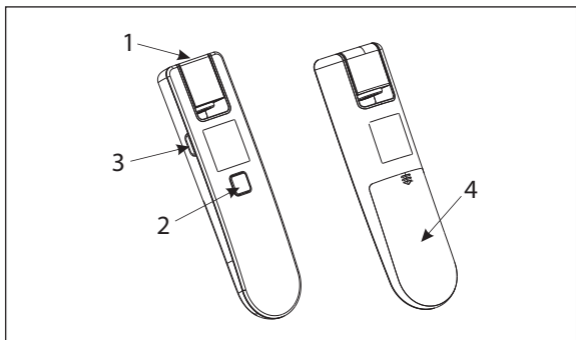

Part Descriptions:

1. Temperature Sensor
2. Start Button
3. Setting Button
4. Batteries

STEP 2

Program Date and Time

After installing the batteries, the temperature unit is flashing. Follow these steps to program the thermometer.



Press the Start button to change the temperature unit and the Setting button to continue.




The time format is flashing (12H or 24H format). Press the Start button to change or the Setting button to continue.



The hour is flashing. Press the Start button to change or the Setting button to continue.



The minutes are flashing. Press the Start button to change or the Setting button to continue.

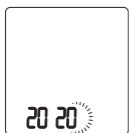

The year is flashing. Press the Start button to change or the Setting button to continue.



The month is flashing. Press the Start button to change or the Setting button to continue.



The day is flashing. Press the Start button to change or the Setting button to continue.

When completed the thermometer will turn OFF.

Note: For additional details on how to program the thermometer, please refer to the instruction manual.

STEP 3

4 Position Rotating Temperature Sensor

This unique temperature sensor allows it to turn in 4 directions.



Towards the user for self measurement.



Towards another person to take their measurement.



Straight up to take a measurement for another person or an object (if in object mode).



Straight down (closed position) which turns the thermometer off.

STEP 4

Take a measurement.

1. Rotate the temperature sensor to the appropriate position, the display will turn ON.
2. Aim the thermometer at the center of the forehead with a distance of less than 2" but do NOT touch the forehead.
3. Press the Start button to take a reading.
4. When done, rotate the temperature sensor downwards and the display will turn OFF.

Note: If the thermometer was too far away from the forehead, the word FAR is shown on the display. Take another reading that is closer than 2" from the forehead.

Frequently Asked Questions

My reading is off by several degrees.

Check if the thermometer is in object mode.

Make sure you are not touching your forehead when taking a measurement.

My device does not turn ON.

Change the batteries (use a brand new set of batteries).

Rotate the temperature sensor to an ON position.

For additional FAQ's, please visit our website at www.zewa.com

Warranty Registration

To register your warranty, visit us online at:

www.zewa.com

For a detailed description of your warranty,
please refer to the instruction manual.

Zewa Inc.

12960 Commerce Lakes Dr. # 29

Fort Myers, FL 33913

Toll Free: 888-993-3592

www.zewa.com